

# Fabius GS as a ceiling device

## Supplement for the Instructions for Use

Please file away in the  
product folder!

## **Intended Use**

A mounting support, to equip the anesthesia device Fabius GS onto a ceiling supply unit in rooms used for medical purposes.

The application of a ceiling mounting support is designated for buildings, not for mobile facilities such as ambulances, helicopters or vessels.

## **For Your Safety and that of Your Patients**

### **Strictly follow the Instructions for Use**

Any use of the device requires full understanding and strict observation of these Instructions for Use. The device is only to be used for the specified purposes.

### **Maintenance**

Repair and general overhaul of the device may only be carried out by trained service personnel. We recommend that a service contract be obtained with DrägerService and that all repairs also be carried out by them. Only original Dräger spare parts may be used for maintenance.

### **Liability for proper function or damage**

The liability for the proper function of the device is irrevocably transferred to the owner or operator to the extent that the device is serviced or repaired by personnel not employed or authorized by DrägerService or if the device is used in a manner not conforming to its intended use.

Dräger cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability conditions for the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

Dräger Medical AG & Co. KGaA

## **Fabius GS as a ceiling device**

### **Characteristics**

In connection with the ceiling supply units DVE 603X or DVE 808X, the anesthesia device Fabius GS can be used as a ceiling device.

The ceiling supply units can lift the anesthesia device Fabius GS by using the „equipment holder M“ and the belonging control unit for the lift.

### **Coupling the Fabius GS onto the DVE**

- For Movita 603X: see Instructions for Use of Movita 600X/603X (order no.: 9037479), chapter „Coupling of the Julian or Mova Cart trolley“. The coupling of the Fabius GS is carried out identically.
- For DVE 808X: see Instructions for Use of DVE 8080/8081/8082 (-C/-M), order no.: 9029046, chapter „Julian or other devices on the ceiling supply unit DVE 808X-M“. The coupling of the Fabius GS is carried out identically.

### **Care**

as with Fabius GS.

These Instructions for Use apply only to  
**Fabius GS as a ceiling device**  
with Serial No.:

If no Serial No. has been filled in by Dräger  
these Instructions for Use are provided for  
general information only and are not intended  
for use with any specific machine or device.



Directive 93/42/EEC  
concerning Medical Devices

**Dräger Medical AG & Co. KGaA**  
Germany

🏠 Moislinger Allee 53 – 55

D- 23542 Lübeck

☎ +49 451 8 82- 0

📠 26 80 70

FAX +49 451 8 82- 20 80

🌐 <http://www.draeger.com>

**90 37 727** - en

© Dräger Medical AG & Co. KGaA

1<sup>st</sup> edition - October 2002

Subject to alteration